


腕關節器械 拆卸和特殊清洗手冊

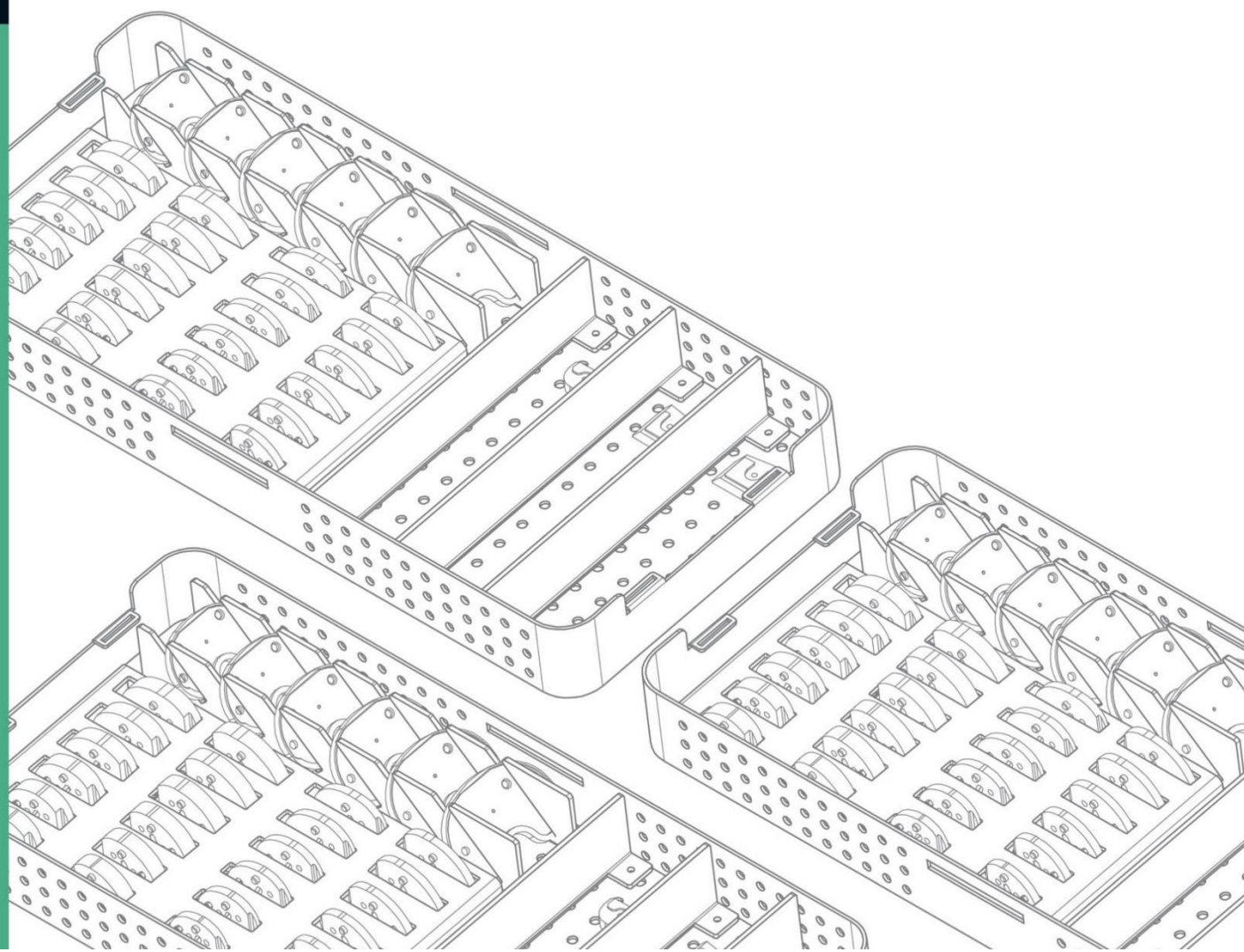

目錄

臨床清洗 - 圖形符號.....	I
股骨銼握把.....	1
EX. 9104-6103-RA	1
髌臼杯擊入器, 偏心型.....	2
EX. 9206-1105	2
全髌臼杯植入器配接件.....	3
EX. 9203-2101	3
髌臼圓紋刀手柄, 直型, 易洗.....	4
EX. 9206-1101	4
髌臼圓紋刀手柄, 偏心型.....	5
EX. 9206-1102	5
髌臼圓紋刀手柄, 偏心型.....	7
EX. 9206-1102-RA	7
EX. 9206-1102-RB	7
雙向偏心型骨銼手柄.....	11
EX. 9106-1501	11
EX. 9106-1502	11
可撓型紋刀轉軸.....	12
EX. 9104-1001	12
全高耐磨聚乙烯髌臼杯定位器.....	13
EX. 9203-1328	13
模組可撓性桿.....	14
EX. 9206-1107	14
無棘輪式手柄.....	15
EX. 9107-1101	15
偏心型骨銼手柄.....	16
EX. 9106-1301	16

股骨柄快速夾持器.....	17
EX. 9104-1214	17
EX. 9106-1111	17
EX. 9106-1311	17
棘輪手柄.....	18
EX. 9721-1041	18
EX. 9721-1041-RA	18
滑錘.....	19
EX. 9101-1203	19
骨柄拔除器.....	20
EX. 9104-1202-RA	20
直股骨銼手柄.....	21
EX. 9106-1101	21
T型手柄.....	22
EX. 9101-1100	22
EX. 9303-1300	22
萬向螺絲起子.....	23
EX. 9206-1114	23
EX. 9601-1213-RD	23

臨床清洗 - 圖形符號

圖示	標題 描述	標準/參考編號
	方向 運動或施力的方向（推・拉和彎曲）	ISO 14617-2 / 241
	雙向 雙向的運動或施力	ISO 14617-2 / 245
	有限的直線運動 有限的直線運動	ISO 7000 / IEC 60417 / 0001
	連續旋轉的方向 順時針旋轉運動	ISO 7000 / IEC 60417 / 0004
	有限旋轉 有限的順時針旋轉運動	ISO 7000 / IEC 60417 / 0006
	雙向的旋轉 順時針和逆時針方向旋轉運動	ISO 7000 / IEC 60417 / 0005
	順時針旋轉移動 與旋轉軸平行的順時針旋轉移動	ISO 7000 / IEC 60417 / 0978
	逆時針旋轉移動 與旋轉軸平行的逆時針旋轉移動	ISO 7000 / IEC 60417 / 0976
	鎖定；緊閉 將兩個零件鎖定或夾緊	ISO 7000 / IEC 60417 / 0018
	鬆鎖；鬆開 將兩個零件鬆開鎖定或夾緊	ISO 7000 / IEC 60417 / 0019
	小心 小心或使用說明	ISO 15223-1 / 5.4.4 EN 980 / 5.11
	水龍頭沖洗 確定水龍頭的開啟或控制開啟供水	ISO 7000 / IEC 60417 / 5574

	噴水 使用注射器、吸移管和/或噴射水流噴洗	ISO 7000 / IEC 60417 / 0073
	旋轉刷洗 使用管刷或試管刷穿過或扭轉運動	ISO 7000 / IEC 60417 / 0307
	鎖定 確定控件功能被鎖定或顯示鎖定狀態	ISO 7000 / IEC 60417 / 5569
	解鎖 確定控件功能未被鎖定或顯示解鎖狀態	ISO 7000 / IEC 60417 / 5570
	製造商 依據歐盟指令90/385/EEC, 93/42/EEC 和 98/79/EC 所述，表明醫療器材製造商	ISO 15223-1 / 5.1.1 EN 980 / 5.12
	歐盟授權代表 標示歐盟的授權代表	ISO 15223-1 / 5.1.2 EN 980 / 5.13
	成功 標示成功的狀態	ISO 7000 / IEC 60417 / 6334A
	失敗 標示失敗的狀態	ISO 7000 / IEC 60417 / 6335B

 有關更詳細的清潔程序，請參閱 UOC-UM-UN-00029 (MT50867)，可重複使用的手術器械的再處理說明





聯合骨科器材股份有限公司
 新竹科學工業園區園區二路57號
 電話：+886-3-5773351
 傳真：+886-3-5777156
 E-MAIL:service@unitedorthopedic.com




mdi Europa GmbH
 Langenhagener Straße 71
 30855 Langenhagen, Germany
 TEL: +49-511-3908 9530
 FAX: +49-511-3908 9539

股骨鋸握把

EX. 9104-6103-RA

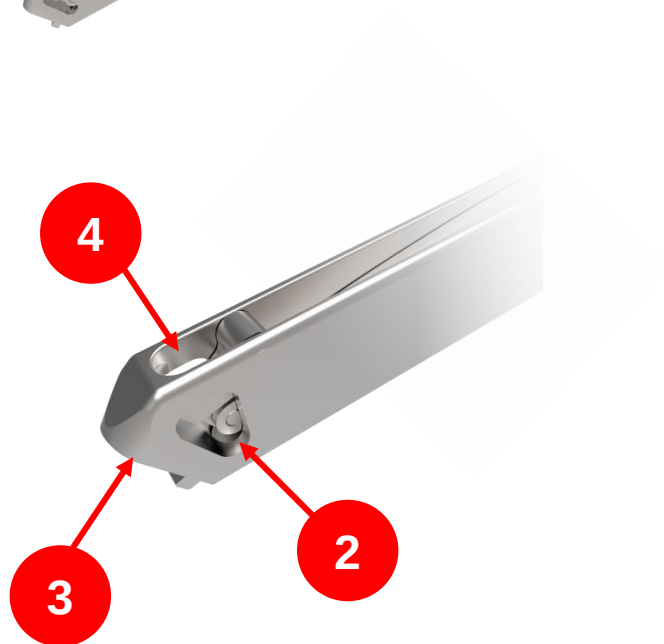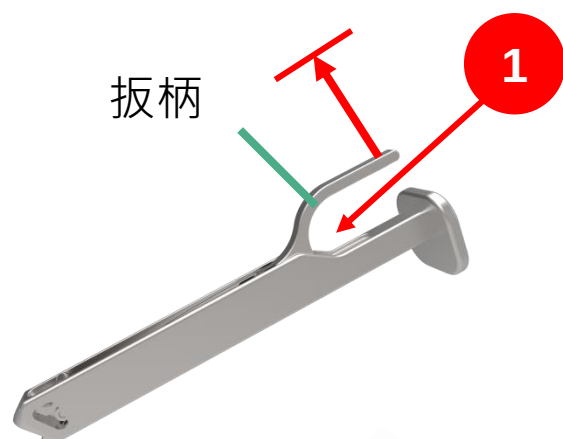







清洗方法

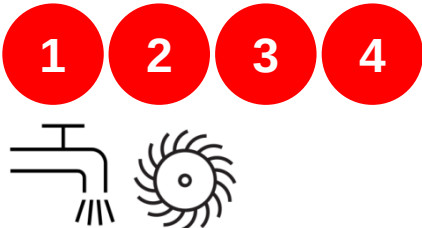

髌臼杯擊入器, 偏心型

EX. 9206-1105

拆卸和組裝

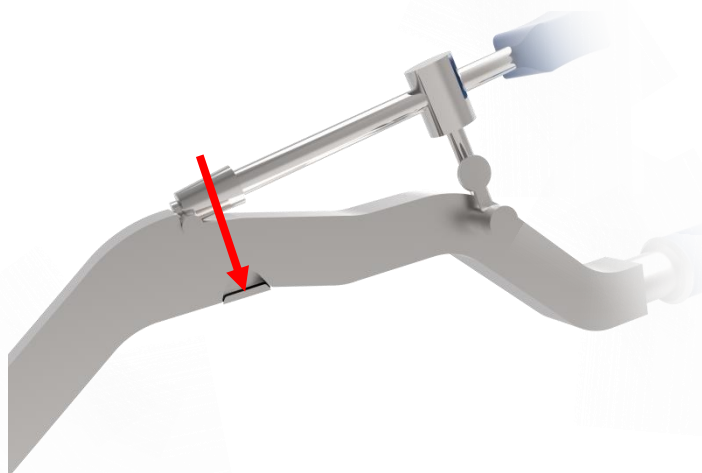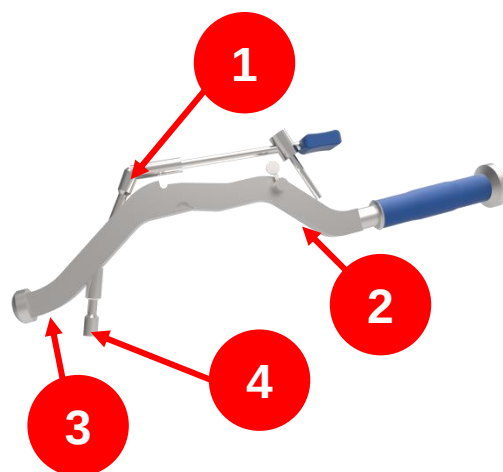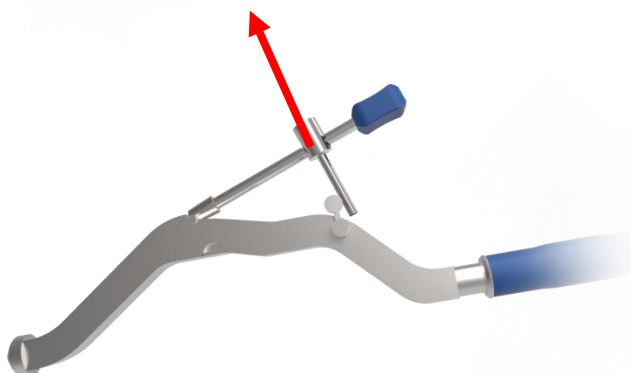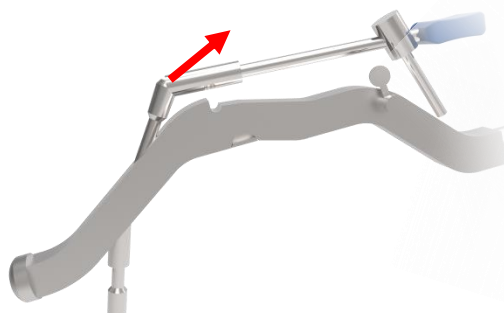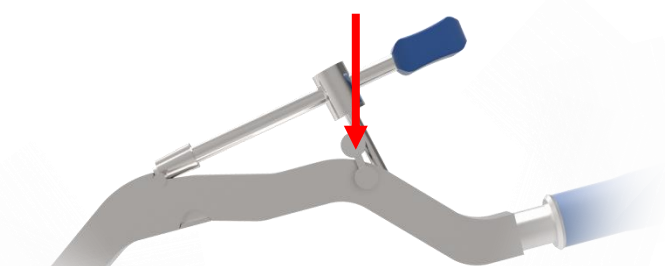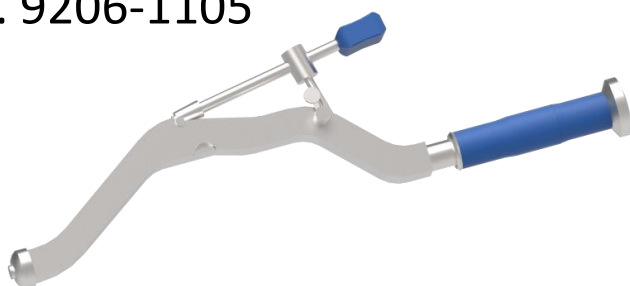


清洗方法




全髖臼杯植入器配接件


EX. 9203-2101

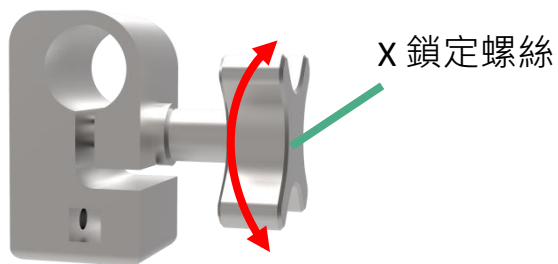

拆卸

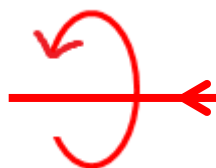

組裝

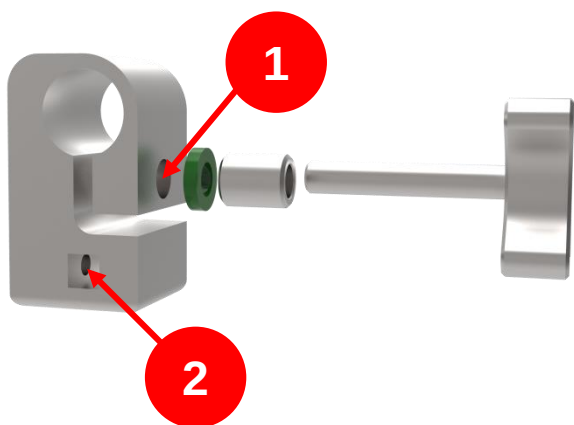

清洗方法





髑臼圓絞刀手柄, 直型, 易洗


EX. 9206-1101



拆卸

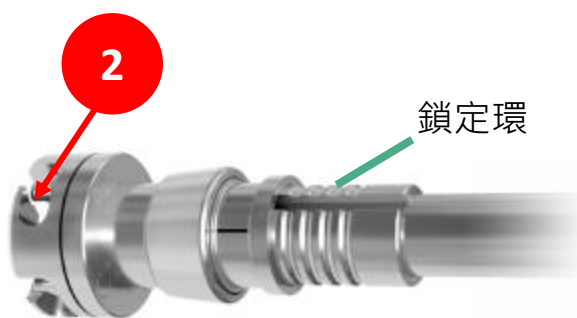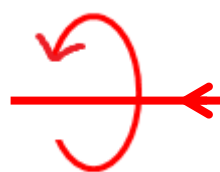

組裝




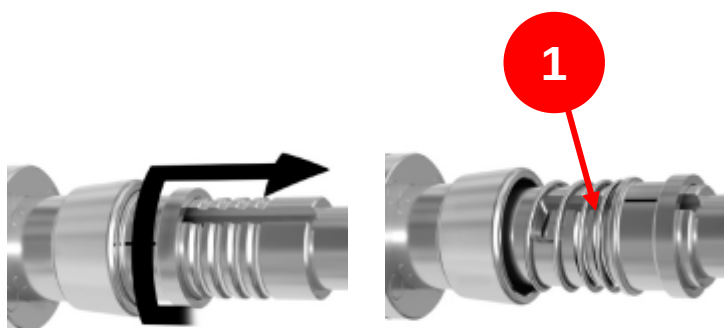


清洗方法




髁臼圓紋刀手柄, 偏心型


EX. 9206-1102

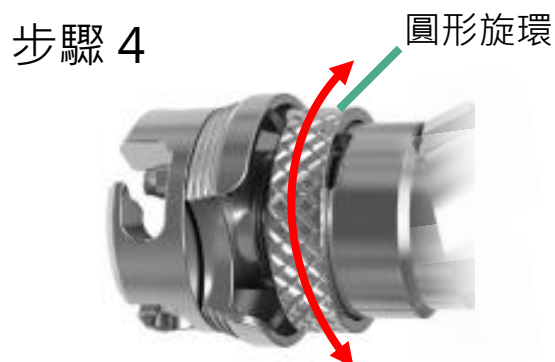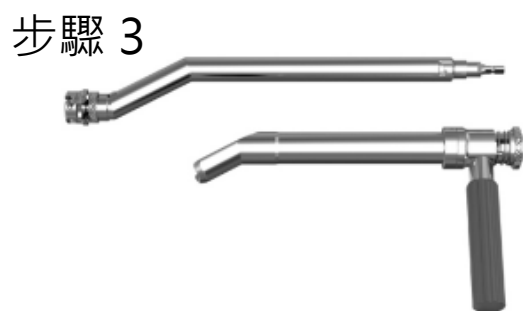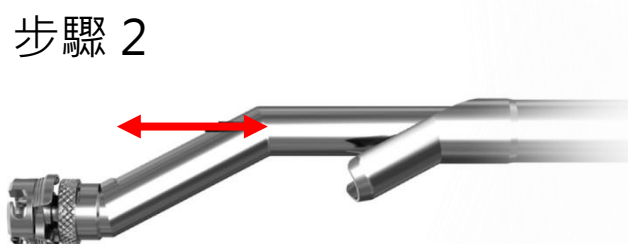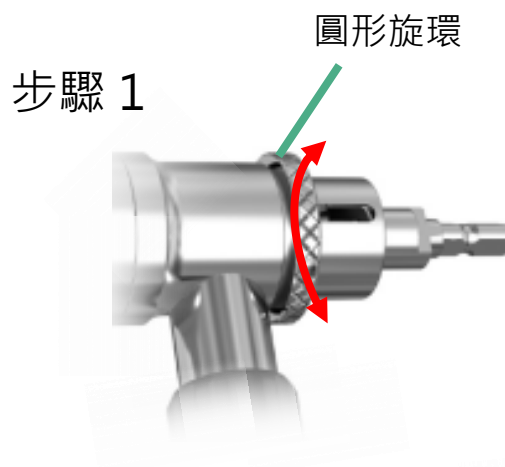

拆卸

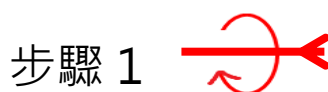

步驟 6

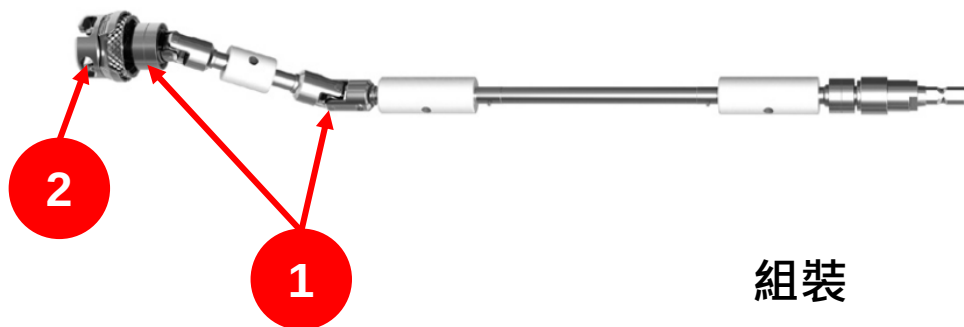

組裝

步驟 1 

步驟 2 

步驟 4 

步驟 5 

清洗方法

1 2



滅菌前須維持拆卸狀態

髯白圓絞刀手柄, 偏心型

EX. 9206-1102-RA


EX. 9206-1102-RB

拆卸

步驟 1




步驟 2

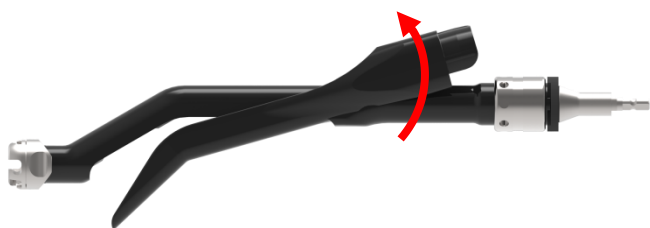


步驟 4




步驟 3

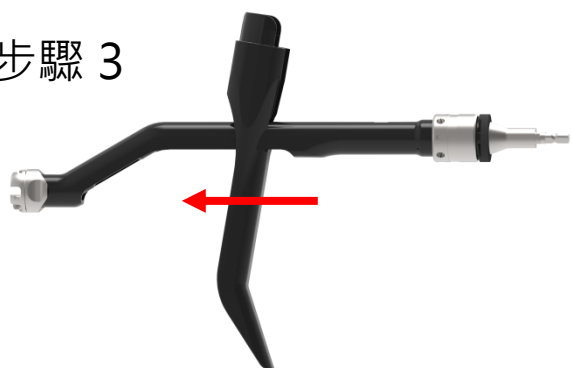


步驟 5

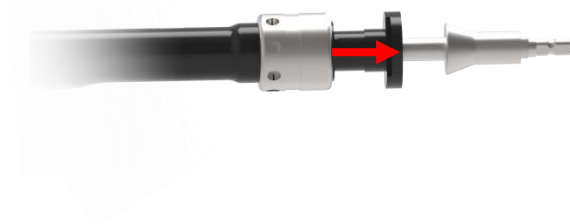


步驟 6

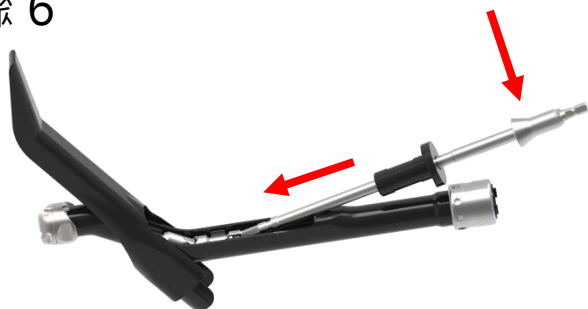


步驟 8

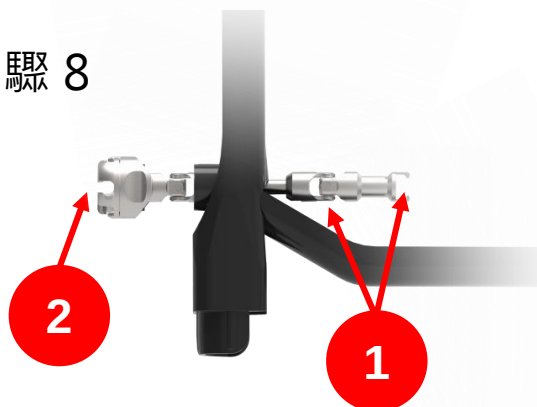

步驟 9

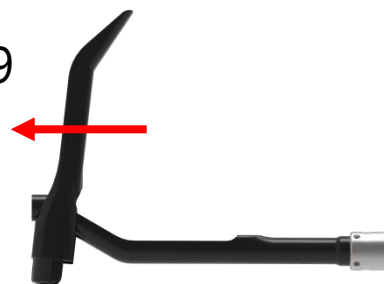

步驟 7

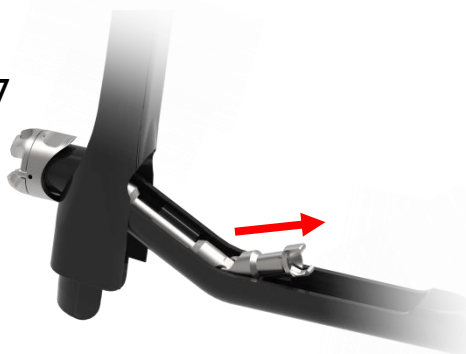


清洗方法



滅菌前須維持拆卸狀態

組裝


步驟 1

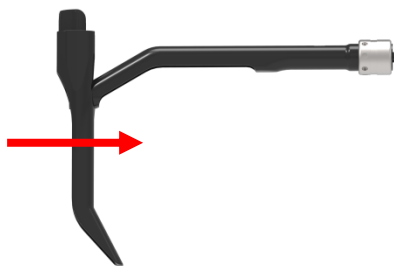


步驟 4




步驟 2

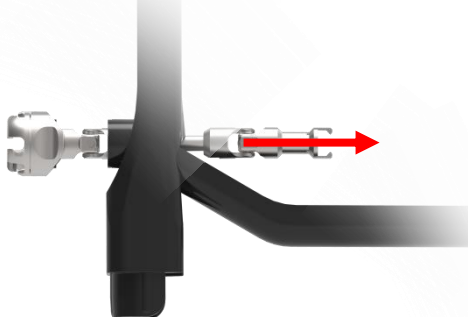


步驟 5

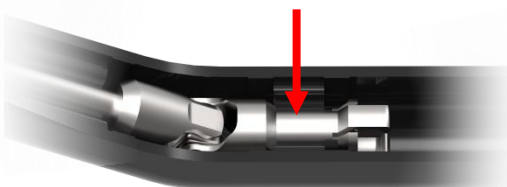


步驟 3

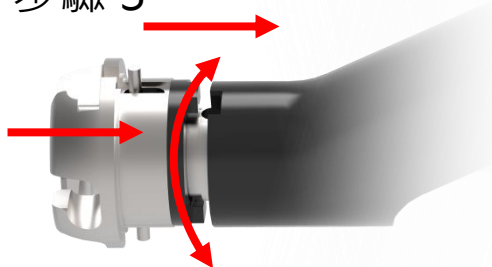


步驟 6

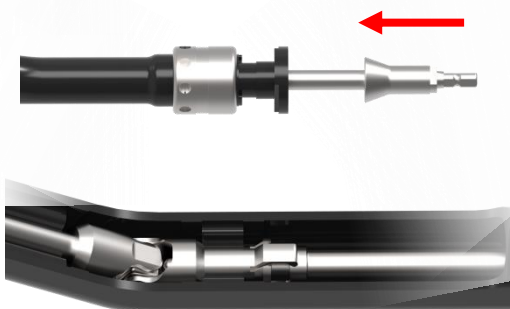

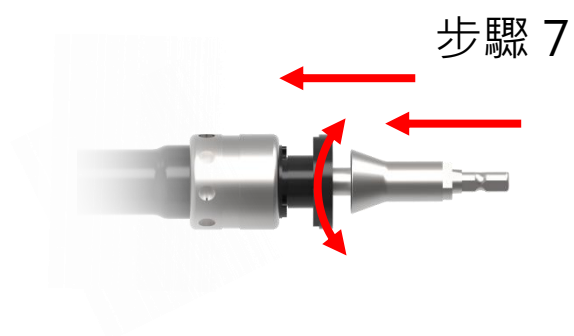


步驟 10

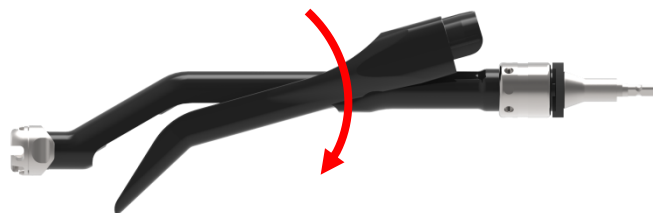


步驟 8



步驟 11

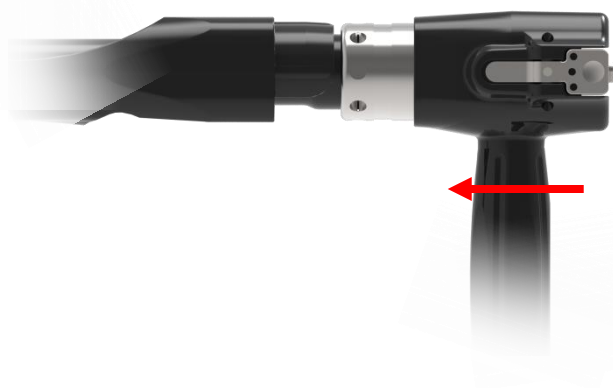


步驟 9

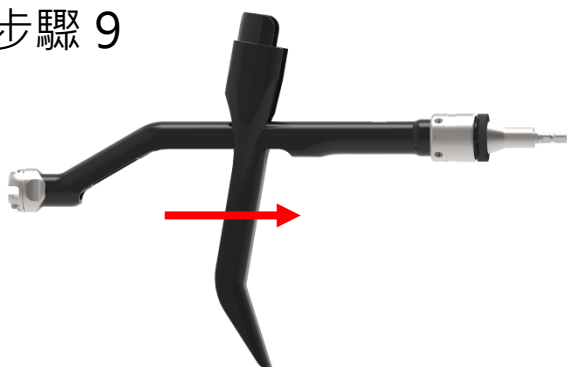

雙向偏心型骨鏟手柄

EX. 9106-1501

EX. 9106-1502

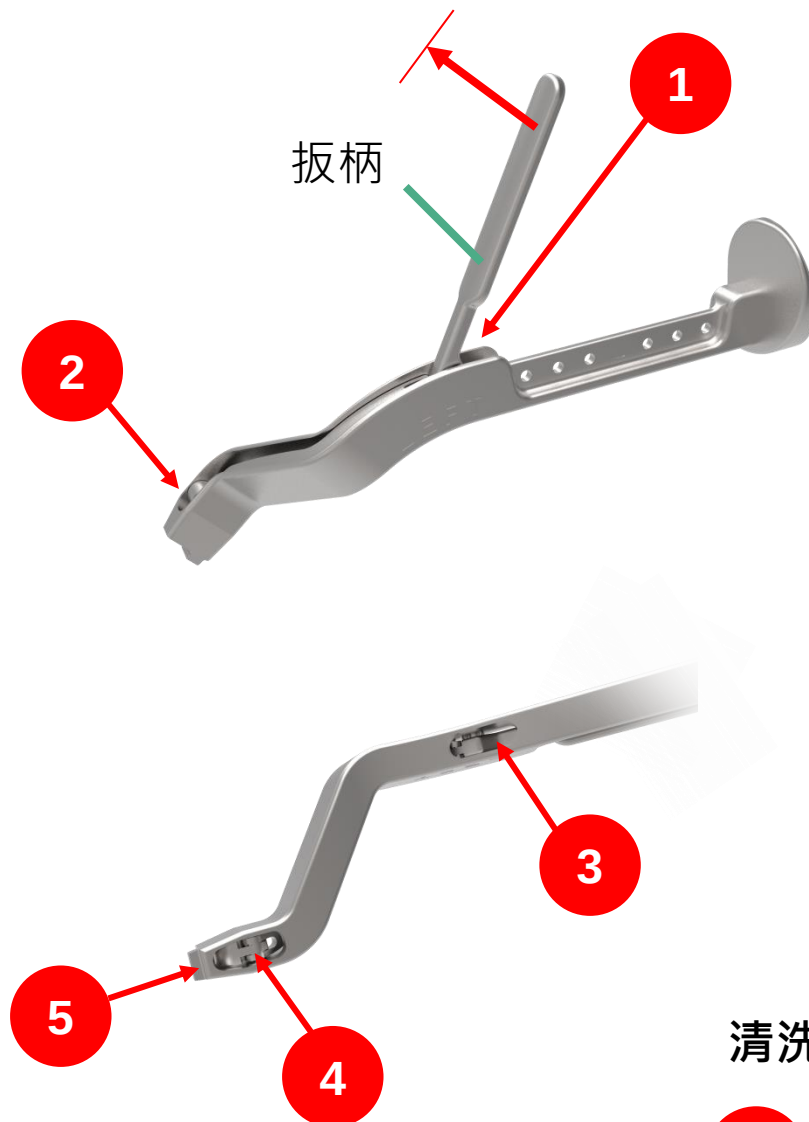


清洗方法






可撓型絞刀轉軸

EX. 9104-1001

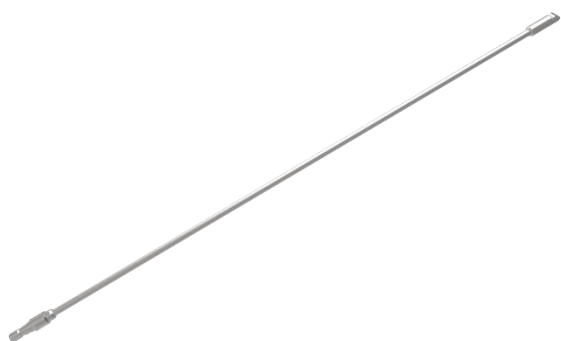


清洗方法

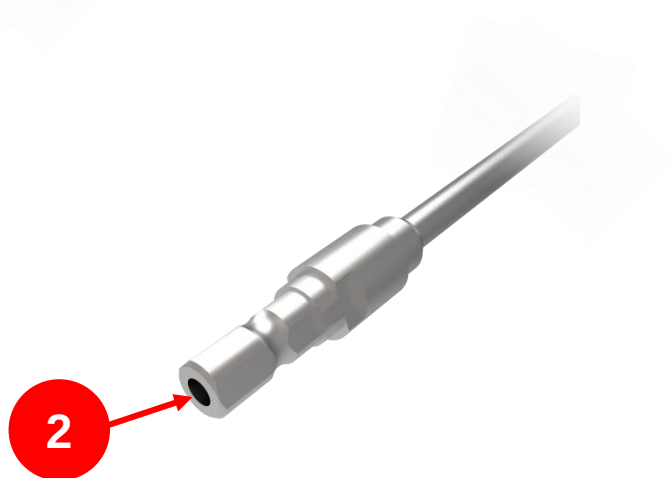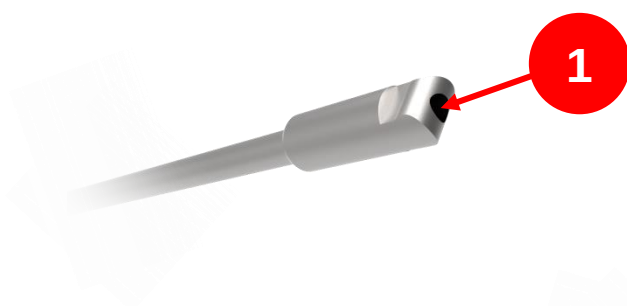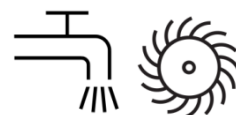


全高耐磨聚乙烯 髌臼杯定位器

EX. 9203-1328

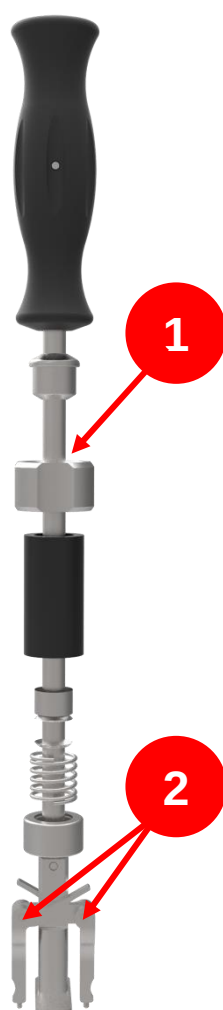


拆卸

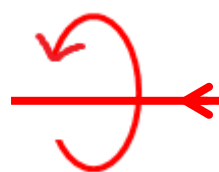

組裝



清洗方法

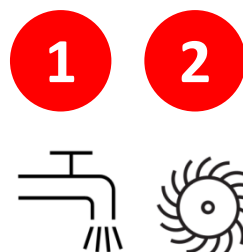


模組可撓性桿

EX. 9206-1107



連接器


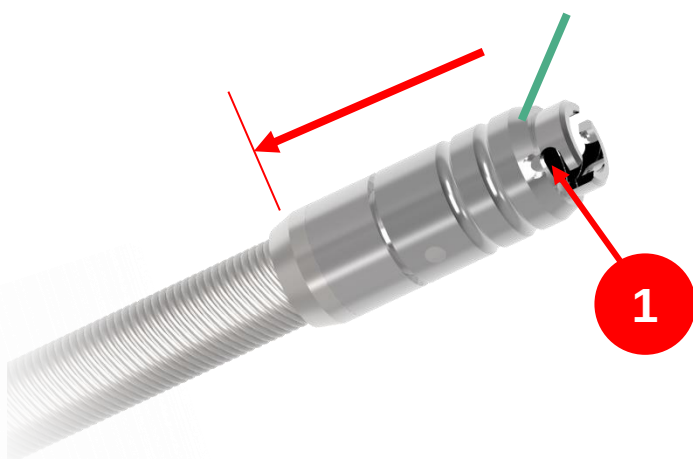


清洗方法

1



2

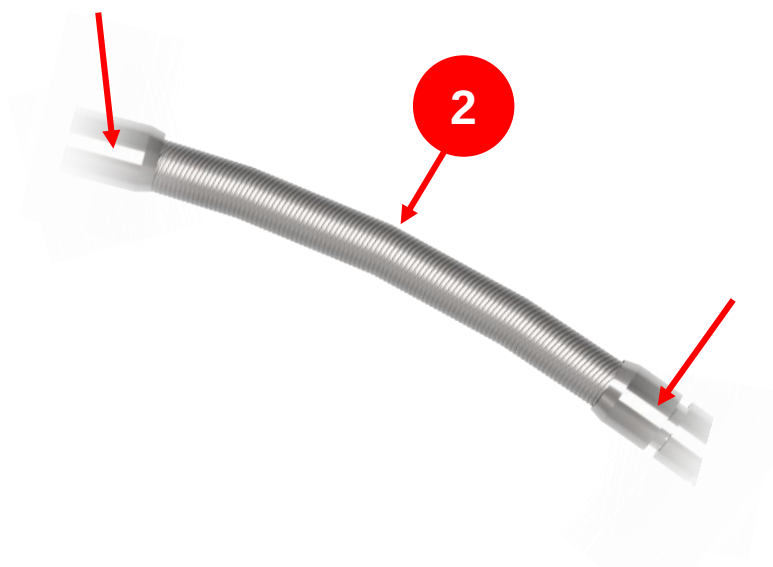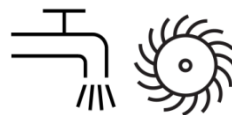

分段局部彎曲撓性桿以沖洗所有縫隙


無棘輪式手柄

EX. 9107-1101



清洗方法

1

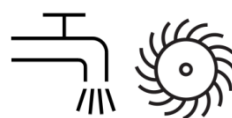


1






偏心型骨鋸手柄

EX. 9106-1301

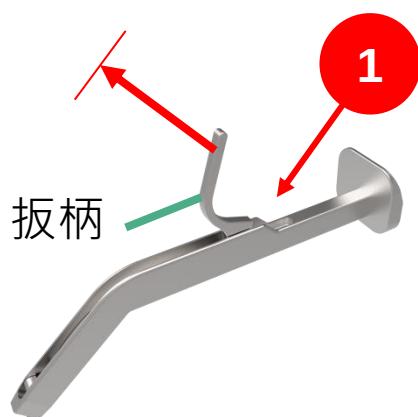

清洗方法

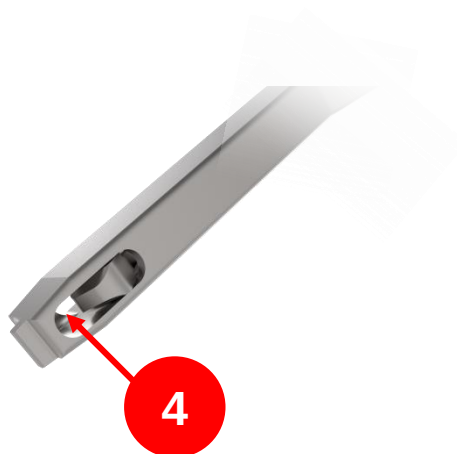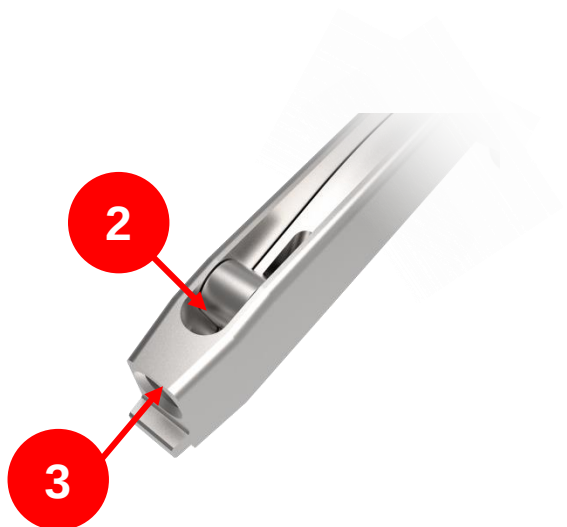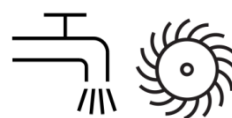


股骨柄快速夾持器



EX. 9104-1214

EX. 9106-1111

EX. 9106-1311

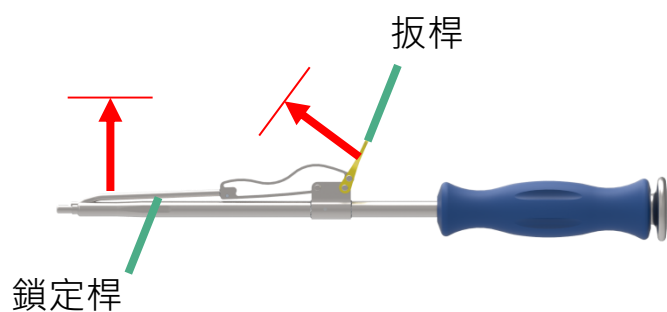

清洗方法

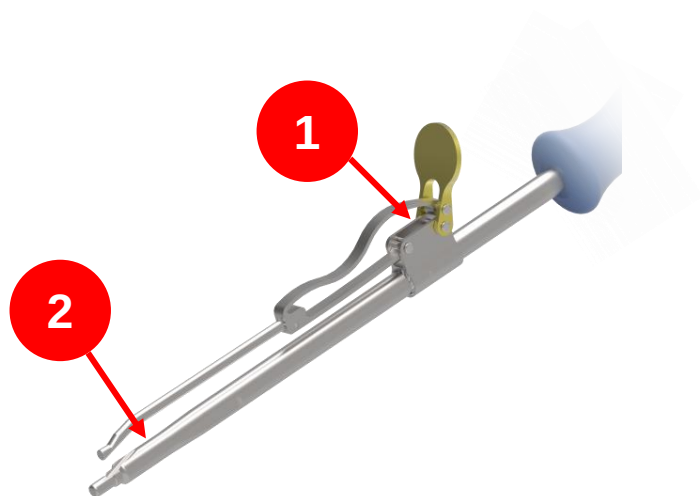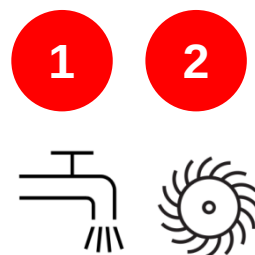

鎖定桿勿分開大於一個手指寬度 (2 公分)



棘輪手柄

EX. 9721-1041


EX. 9721-1041-RA



清洗方法

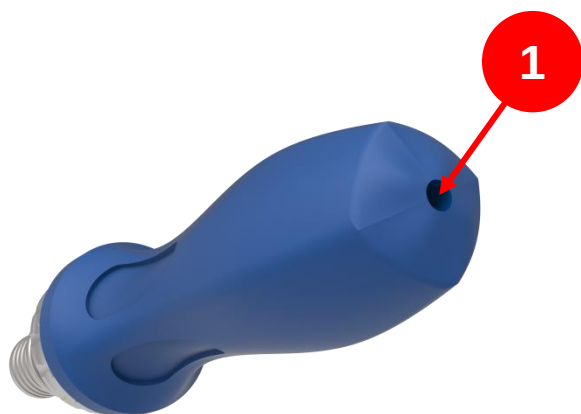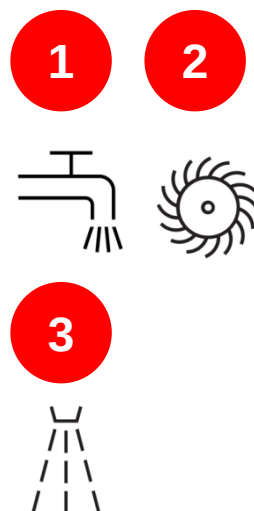


連接套筒

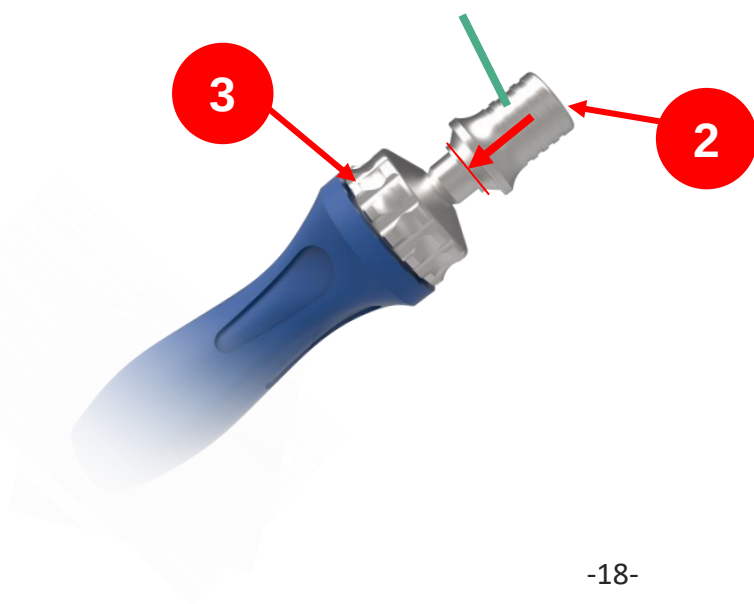


滑錘

EX. 9101-1203

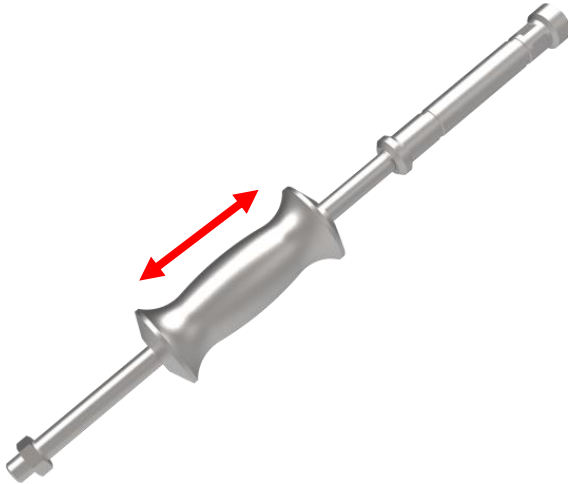


清洗方法

1

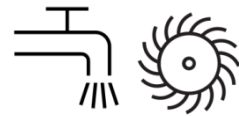


1

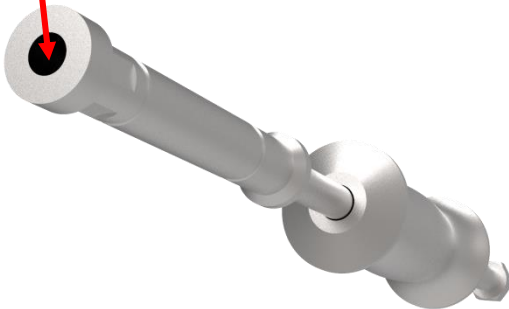


骨柄拔除器

EX. 9104-1202-RA


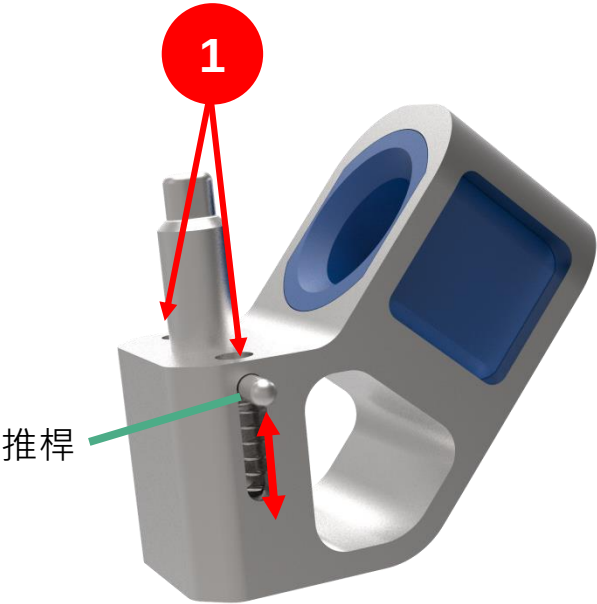


清洗方法

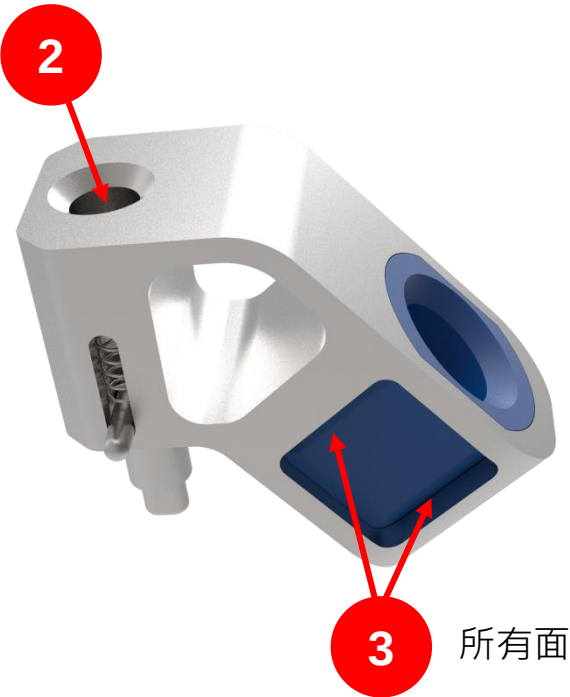

直股骨鋸手柄

EX. 9106-1101

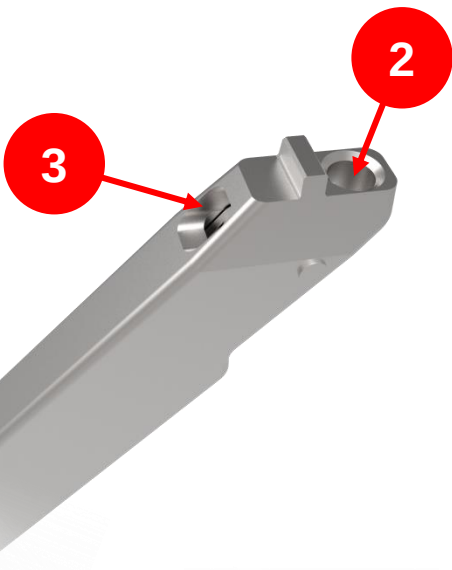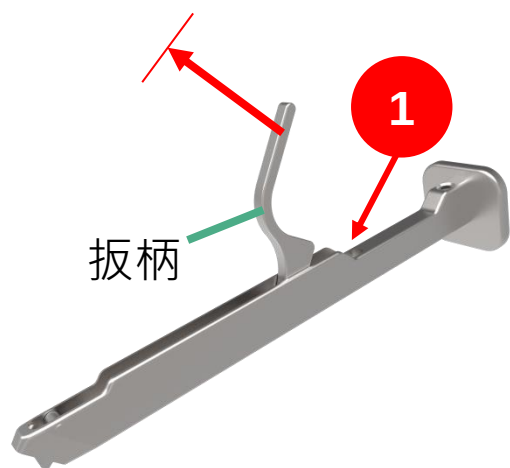


清洗方法



T 型手柄


EX. 9101-1100

EX. 9303-1300

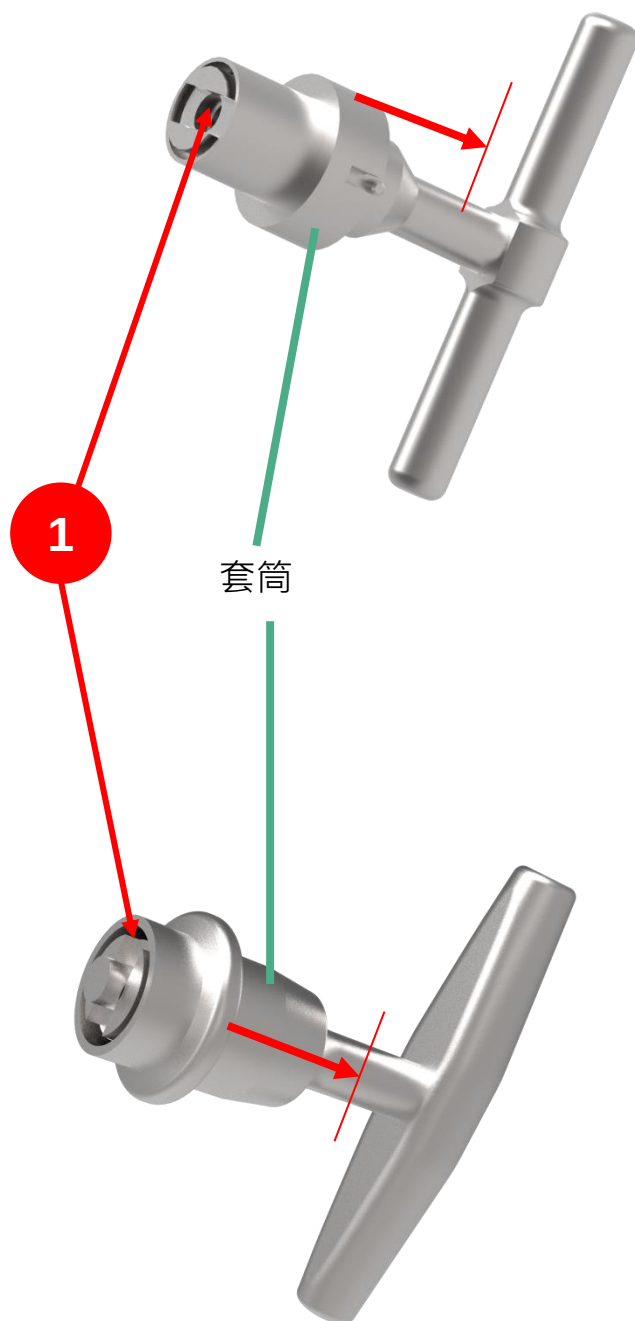

清洗方法

1

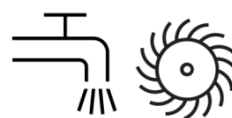


萬向螺絲起子

EX. 9206-1114

EX. 9601-1213-RD



鉸鏈結構

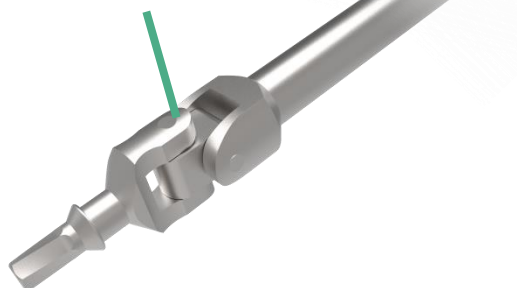


清洗方法

1



1

雙面之間

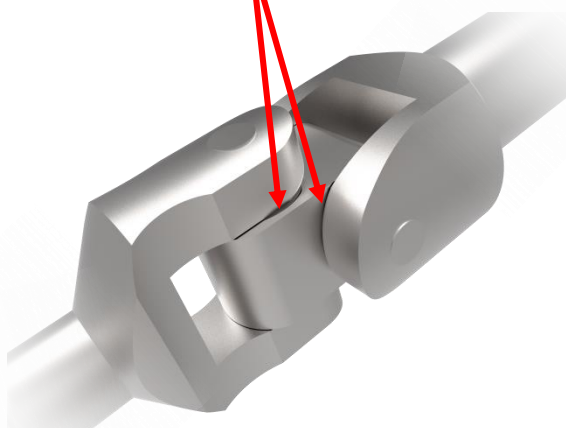

Please note that this Surgical Technique Guide has been authored in the English language. Any translations into other languages have not been reviewed or approved by United Orthopedic Corporation and their accuracy cannot be confirmed. Any translated guide should be reviewed carefully prior to use and questions regarding a Surgical Technique Guide should be directed to United Orthopedic Corporation at unitedorthopedic.com/contact

The CE mark is valid only if it is also printed on the product label.

